# **Simple Maintenance Procedures Extend the Life of Your Handpieces**

### **OneStep** Handpiece Lubricant

HAYES

REPAIRS | SALES | TRAINING



- Longer turbine life!
- Available in aerosol, dropper bottle or bottle for automatic lubrication station

### **Hayes Automatic** Lubrication **Station**



- Easy & efficient
- Quickly lube 2 high-speeds and 1 E-type handpiece

## **Tidy Whities**

**Original Handpiece** & Instrument Wipes





### **HIGH-SPEED HANDPIECES**

**Clean and lubricate** 

Scrub and rinse handpieces with mild soap and water or wipe with Tidy Whities<sup>™</sup> original handpiece & instrument wipes before lubricating. (Do not wipe down with cleaning/disinfecting wipes or other harsh chemicals.)

### **AEROSOL INSTRUCTIONS:**

- A. Remove bur
- B. Select the appropriate nozzle and insert into swivel back-end or back of 4/5-hole handpiece Selecting the proper nozzle will ensure a tight seal to propel lubricant throughout the handpiece
- C. Spray one second when using Hayes OneStep aerosol (Refer to manufacturer instructions for J. Morita high-speeds)
- D. Clean chuck weekly with lube and proxy brush



#### Run for 20 seconds. Handpieces requiring a bur tool must be run with the bur



**AYES** 

### **Between-Patient/Weekly Manual Maintenance for**


### LOW-SPEEDS, **ATTACHMENTS & SCALERS**


### **Clean and lubricate**

 $\bigcirc$ When applying the lubricant into a 4/5-hole handpiece with dropper bottle, use the drive air tube only. Place 1-3 drops from the dropper bottle at all points as indicated below. For motors, perform weekly, as needed, or as specified by the manufacturer. For attachments and heads, perform between patients. Clean threads


monthly with 2x2 and alcohol or Tidy Whities wipes.

#### LOW-SPEED MOTORS & AIR SCALERS

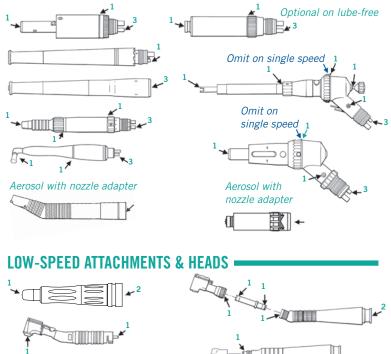

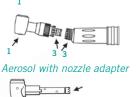

3

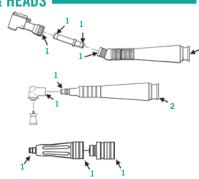

Drive Ail

HAYES

### ELECTRIC, ENDO & IMPLANT HANDPIECES



Lubricating and flushing electric handpieces between patients is crucial. However, never lubricate an electric motor. Please consult your Haves office for detailed instruction

- Cleans & shines
- Convenient
- Non-corrosive

### **Treat Your** Waterlines!



- Remove microbial contamination from waterlines and bottles
- Protect staff, patients, and handpieces!



installed. If flush station is not installed, run handpiece chairside. Wipe handpiece exterior with paper towel.

We sell the DCI Flush Station! Call Hayes for more information.

### **Bag and sterilize**

Always let the handpieces go through the complete drying cycle (never use dry heat). Handpieces are ready to use and DO NOT NEED TO BE **RELUBRICATED.** Never sterilize with a bur installed.

### **Run the handpiece**

Run for 20 seconds. Wipe handpiece exterior with paper towel.

### **Bag and sterilize**

Always let the handpieces go through the complete drying cycle (never use dry heat). Handpieces are ready to use and DO NOT NEED TO BE **RELUBRICATED.** Never sterilize with a bur installed.

HAYES

REPAIRS | SALES | TRAINING

www.hayeshandpiece.com

Call us to purchase maintenance products or to schedule staff training!

> **FACTORY-TRAINED** Bien Air Miller MIDWEST StarDental 3